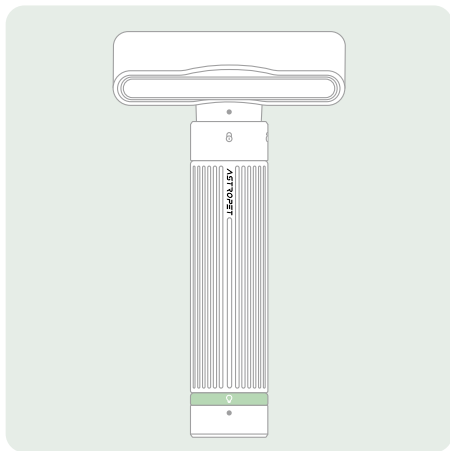
POWERED BY

ASTROPET

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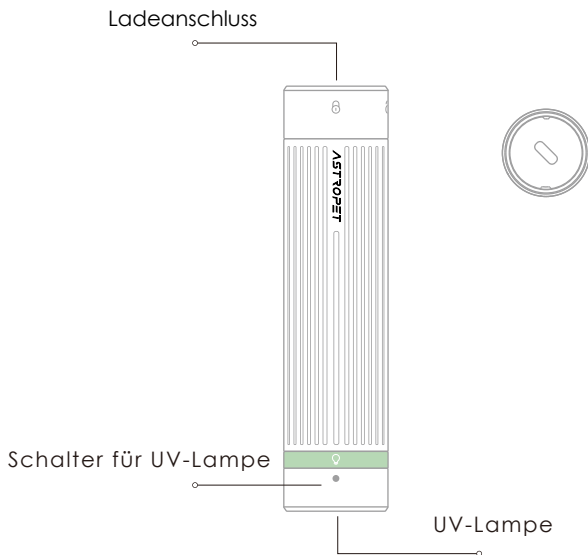
Pet Brush with UV Lamp

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EN User Manual

1 Produktinformation

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Hinweis:

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

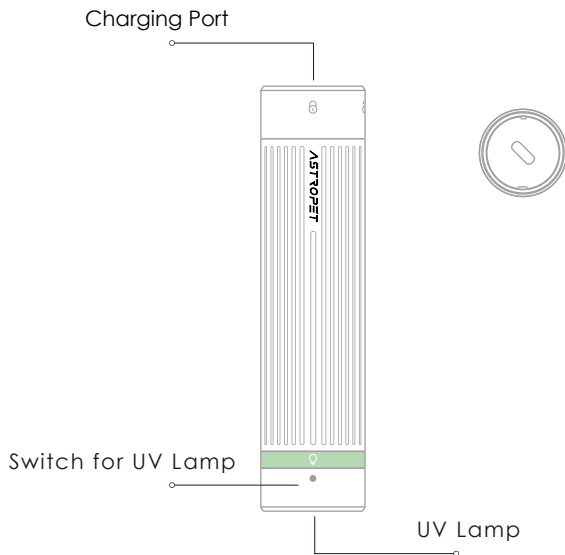
② Packing List

Notice:

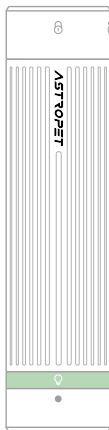
After removing it from the package, please ensure that the product is in good condition and compare the accessories with the instruction manual. Please note that due to product updates and iterations, the accessories may have undergone adjustments or changes. The actual objects should be considered as the final reference.

Handle

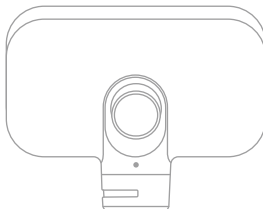Pinstripe Comb

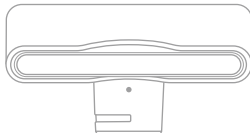
Hair Removal Comb

3 Instructions

Comb head replacement

Remove the comb head by rotating it upward as indicated by the handle pattern ①.

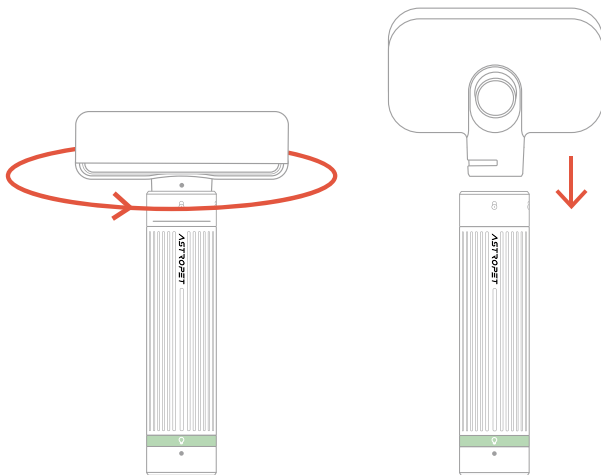To install, align the tabs on the comb head with the pattern ① and insert, rotating until ② installation is complete.

UV Lamp (Wood's lamp) Examination

To activate the UV lamp, rotate the handle at the rear as indicated by the diagram. During use, ensure the examination takes place in a moderately dimly lit room. Direct the lamp's light onto the suspected affected area of the pet's skin. Illuminate the affected area in a darkened room.

In 50 - 60% of *Microsporum canis* infections there will be greenish fluorescence which runs along the hair shafts. Observe the fluorescence emitted to assess the potential presence and severity of dermatophytosis in the pet's skin.

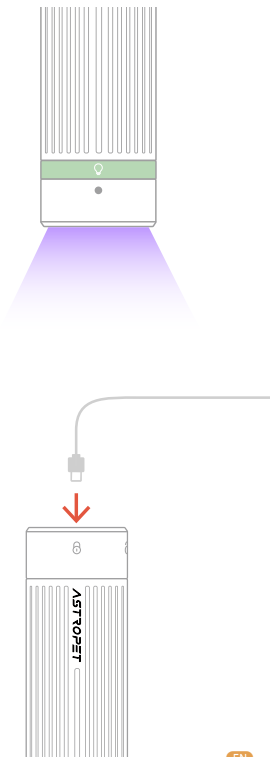
Important Notice:

*The results obtained from this examination should serve as references only and cannot be considered conclusive for making a diagnosis. For a definitive diagnosis, please visit a veterinary clinic for a professional consultation.

*Drugs, soaps and bacteria (*Pseudomonas* sp.) or occasional individual scales can fluoresce as well, but they should not be associated with the hair shafts. Be careful: A lack of fluorescent areas does not rule out dermatophytosis.

Charging

Remove the comb head as described above and connect the type-c power cord for charging, a red light indicates charging is in progress and a green light indicates charging is complete.

4 Product Specifications

Product name: Pet brush with UV lamp
Main material: ABS, TPR, Stainless steel
Packing list: Handle x 1, Comb head x 2

Battery capacity: 700mAh
Input rating: 5V1A

5 Warranty Card

Dear customers, thank you for purchasing this product, please read the following warranty terms carefully before using this product:

- ① The warranty period is two years, starting from the date of purchase (based on valid purchase records). We provide free maintenance for any quality problem within this duration.
- ② The following conditions are not covered by the warranty:
 - Any malfunction or defect caused by improper conditions of use of environment;
 - Any fault or defect caused by improper installation, misuse, unauthorized maintenance, force majeure events, or other external reasons;
 - Any fault or defect caused by use of non-original fittings or accessories;
 - The product has exceeded the warranty period;
 - Other failures or damages caused by none of product design, technology, manufacturing process or quality problems;
 - Normal fading, wear and tear or consumption during the use of the product;
 - Not able to provide valid purchase certificate.
- ③ Please contact Customer Service within a proper time to confirm warranty service situation.
- ④ If the returned product does not meet the valid warranty terms, we will charge you for the corresponding maintenance costs.
- ⑤ Please fill in the after-sales information feedback form and send it together with the returned product.

Feedback

Customer Name:
Your Phone Number:
Your E-Mail:
Your Address:

Product Model:
Date Of Purchase:
Where You Purchase:
Fault Feedback: